

Mi-Rust Remover

Surgical Instrument Rust Remover



CE

Active ingredients :

Phosphoric acid	38% w/v
Citric acid	15% w/v
Malic acid	2% w/v
Oxalic acid	1% w/v
DM water to make	q.s.

- Removes Rust, Stains
- Restores instruments original finish
- Resotres articulation to box- locks and joints
- Prevents destructive pitting on ridges of micro surgical and ophthalmic instruments
- Enhances the life and efficiency of stainless steel
- Eco- friendly, stable, mild acidic



Mi- Rust Remover is a quick, safe and efficient revitalizing instrument Solution

Mi-Rust Remover

Surgical Instrument Rust Remover

Mi-Rust Remover is highly effective cleaner to remove tarnish loose rust and rusted stain.

It also removes stubborn stains, rust stains & organic residues from Instruments. It is an ideal cleaning agent for surgical instruments made from stainless steel.



Areas of Application:

- CSSD
- OT (where CSSD) is not a separate department, .
- Dental units
- Endoscopes
- Surgical instruments
- All stainless steel equipments



Direction for use:

- Mix 10 to 15 ml Mi-Rust Remover in 1 liter of water and immerse for 20 minutes. Wash-off with soft water. Dry thoroughly.

Precautions:

- For external use only.
- Avoid direct contact with eyes and skin, rinse with plenty of water in case of contact. Harmful, if inhaled or swallowed. Wear personal protective gear before handling.
- Keep out of reach of children. Seek medical help in case of emergency.



FAITH MICRO SOLUTIONS

An ISO 9001:2008 Company

Head Office: E 107, Lajpat Nagar-I, New Delhi-110 024 INDIA

Customer Care : info@faithmicrosolutions.com

Authorised Representative: